




**PORTABLE MEDICAL OXYGEN THERAPY KIT
COMBILITE®**

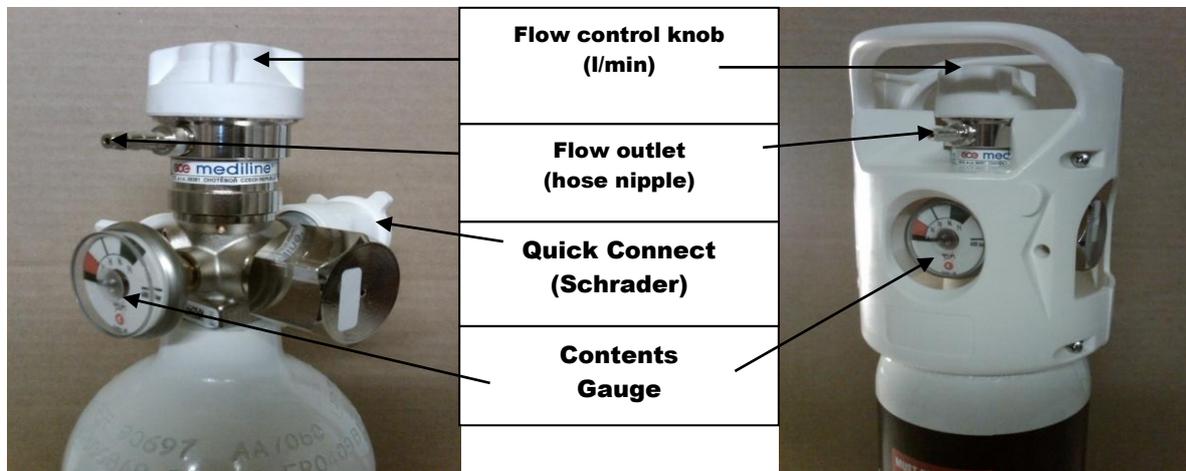

Pre operating instructions:

- 1. Check cylinder has batch label with a valid expiry date.**

Operating instructions:

- 1. Check cylinder contents gauge and ensure there is a sufficient supply of oxygen.**
- 2. Connect therapy tube and mask to the flow outlet(hose nipple) or connect Schrader to the quick connect where fitted.**
- 3. Turn on Flow Control knob slowly in an clockwise direction to turn on the oxygen supply and the desired flow setting (i.e. 0,1,2,3,4,5,7,8,10,12,15)**
- 4. After operation turn off the Flow Control knob.**

IMPORTANT NOTE: This Medical equipment must only be operated by trained, competent personnel.

SAFETY GUIDELINES

Industrial Pressure Testing Ltd, Unit H5, Marina Commercial Park, Centre Park Road, Cork.
[Tel:021-4968383](tel:021-4968383) Fax: 021-4968059 email:indpt@indigo.ie



PORTABLE MEDICAL OXYGEN THERAPY KIT
COMBILITE®

- **Smoking, naked lights, electrical equipment and any source of heat or combustion must not be allowed within the vicinity of the cylinders.**
- **Use no oil or grease on the cylinder or regulator. In particular, make sure that hands are adequately dried after the use of alcohol gels.**
- **Do not drop the cylinders as they contain high pressure oxygen.**
- **Cylinders must not be repainted, have any markings obscured or labels removed.**
- **Do not store fully charged cylinders in an environment where they are exposed to high temperatures, i.e. a car luggage boot.**
- **Warning Notices prohibiting smoking or naked lights must be clearly posted in the storage areas.**
- **If you have any doubts regarding the condition of the device for its intended use withdraw from service immediately and return to Industrial Pressure Testing Ltd for investigation.**
- **Keep the device out of sight and reach of children.**
- **Do not use any tools or wrenches on any part of this device.**
- **If a leak develops do not use force to try and correct the fault, if possible turn off the cylinder, otherwise place in a ventilated area and contact Industrial Pressure Testing Ltd.**
- **No liability will be accepted for loss of oxygen from the cylinder due to improper use by non-trained personnel.**
- **It is important to understand the following. This medical equipment **MUST** only be operated by trained, competent personnel.**